

**GB**      **INSTRUCTION FOR USE**  
**VERISTERIL POUCHES AND REELS**

**1. INSTRUCTIONS FOR USE**

Use VERISTERIL pouches and reels to pack medical devices to be sterilized by steam or EO gas. VERISTERIL with 3 indicators also can be sterilized by formaldehyde.

Seal the open end(s) with a rotary or impulse sealing machine at the recommended temperature (150-180 °C) and following the instructions of the sealing machine manufacturer.

**2. ESTERNAL INDICATORS PRINTED ON THE PRODUCTS:**

VERISTERIL:  
EO gas: Yellow/Brown sterilized  
Steam: Brown sterilized  
VERISTERIL with 3 indicators:  
EO gas: Yellow/Brown sterilized  
Steam: Brown sterilized  
Formaldehyde: Green sterilized

The packaging is for single use only.  
Do not use if pack is damaged.

Report any serious incident involving the device to the manufacturer and the competent authority.

**3. STORAGE CONDITIONS**

Keep away from direct sunlight and heat. We recommend:

- Store at temperature 10-30 °C before use.
  - Relative humidity: 30-60 % before use.
- Outer cartons should not suffer physical damage in handling and storage.

**EN 868-5    ISO 11607-1**



**STERIFIT S.r.l.**

Via dei Martinitt, 3  
20146 Milano - Italia  
info@sterifit.com – www.sterifit.com