# Surface Disinfectant Wipe

For the surface disinfection of non-invasive medical devices.  
Specially formulated for use in the Medical & Dental environment.

**Directions for use:** 1. Remove cap of dispenser tube, 2. Cut open air tight plastic pouch across the top of the pouch, 3. Feed the centre wipe through the hole in the cap and replace, 4. Pull wipe at a 45° angle. Using a new wipe for each application wipe objects to be disinfected so that they are completely moistened and let the disinfectant take action for a minimum of 60 seconds (to achieve full efficacy leave for 5 minutes). Allow surface moisture to evaporate. Do not use on surfaces sensitive to alcohol. For professional use only.

**Efficacy:** Bactericidal (60 secs), including MRSA, VRE. Yeasticidal (60 secs). Mycobactericidal (60 secs). Tuberculocidal (60 secs). Fungicidal (5 mins). Viruses to include (60 secs): Non-enveloped viruses: Murine norovirus, Adenovirus 5. Enveloped viruses: Hepatitis B virus (HBV), Hepatitis C virus (HCV), Hepatitis Delta virus (HDV), Human Immunodeficiency Virus (HIV), Influenza Virus (human, avian and porcine), Coronavirus, Herpesviridae, Filoviridae (Ebola), Rubella, Rabies, Measles.

**Tested to Standards:** EN 14885, EN 14476, EN 14348, EN 13727, EN 13624.

**Composition:** Ethanol, Aqua, Parfum, Linalool, Limonene.

**Danger:** H228: Flammable solid. H319: Causes serious eye irritation. H412: Harmful to aquatic life with long lasting effects. P210: Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. P403+P233: Store in a well-ventilated place. Keep container tightly closed. P280: Wear protective gloves/protective clothing/eye protection/face protection. P303+P361+P353: If ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water. P235: Keep cool. P305+P351+P338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P370+P378: In case of fire: Use alcohol resistant foam or carbon dioxide to extinguish.

**Peril:** P280: Use protective equipment. P303+P361+P353: If ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water. P235: Keep cool. P305+P351+P338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P370+P378: In case of fire: Use alcohol resistant foam or carbon dioxide to extinguish.

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**Testé aux normes:** EN 14885, EN 14476, EN 14348, EN 13727, EN 13624.

**Composition:** ethanol, aqua, parfum, linalool, limonene.

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