3M ESPE**RelyX™ Unicem 2 Automix****Self-adhesive Resin Cement**
Instructions for Use
Gebrauchsinformation
Mode d'emploi

3M ESPE AG
Dental Products
D-6222 Seefeld - Germany

3M ESPE
Dental Products
St. Paul, MN 55144-1000 U.S.A.

3M ESPE Customer Care/MSDS Information in U.S.A., and Canada - 1-800-634-2249.
3M, ESPE, Cojet, Lava, RelyX and Sof-Lex are trademarks of 3M or 3M ESPE AG.
© 3M 2010. All rights reserved.

Informazioni per l'uso
Información de uso
Instrucciones de uso

012

4400759132/01

ENGLISH

Product Description
RelyX™ Unicem 2 Automix, manufactured by 3M ESPE, is a dual-curing, self-adhesive restorative system. It is used for the adhesive cementation of indirect all-ceramic, composite, or metal restorations and for posts and screws. When RelyX Unicem 2 Automix is used, bonding and conditioning of the tooth structure are not necessary.

Unlike other cementation materials, RelyX Unicem 2 Automix is characterized by high stability in combination with good flowability under pressure (structural viscosity).

The cement is available in various shades.

RelyX Unicem 2 Automix contains bi-functional (methacrylate). The proportion of inorganic fillers is about 70% by weight; the grain size (D90%) is about 12.5 µm.

The mixing ratio, based on volume, is 1 part base paste : 1 part catalyst.

Use: These instructions for Use must be kept for reference for the duration of product use. Please see the pertinent information for use for details on all of the products mentioned below.

Indications:

- Final cementing of all-ceramic, composite, or metal inlays, onlays, crowns and bridges; 2-3 unit Maryland bridges and 3-unit inlay/onlay bridges (excluded for patients with bruxism or periodontitis)
- Final cementing of posts and screws.
- Final cementation of all-ceramic, composite, or metal restorations on implant abutments

Indications such as Maryland and inlay/onlay bridges (resin bonded bridges) demand especially high adhesive bond strength. Regardless of the manufacturer of the cement and restoration, these indications may be exposed to a high risk of decementation. To achieve an optimal setting when using RelyX Unicem 2 Automix, please refer to the sections "Tooth Preparation" and "Pre-treatment of Maryland and Inlay/Onlay Bridges".

Precautionary Measures

For Patients and Dental Personnel

• Base paste contains substances that may cause severe eye damage. Wear eye protection to prevent injury. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.

• Base paste: contains sodium persulfate that may trigger an allergic respiratory reaction in certain individuals. This product may not be appropriate for use in those individuals with known sensitivity to sulfites since a cross-reaction may occur with sodium persulfate.

For Patients

- This product contains substances that may cause an allergic reaction by skin contact in certain individuals. Avoid use of this product in patients with known acrylate allergies.
- If prolonged contact with oral tissue occurs, flush with large amounts of water. If allergic reaction occurs, seek medical attention as needed, remove the product if necessary and discontinue future use.

For Dentist Personnel

- This product contains substances that may cause an allergic reaction by skin contact in certain individuals. To reduce the risk of allergic response, minimize exposure to these materials. In particular avoid exposure to uncured product. If skin contact occurs, wash skin with soap and water.

• The use of protective gloves and a no-touch technique is recommended.

Acrylates may penetrate gloves easily. If the product contacts the glove, remove and discard the glove, wash the hands immediately with soap and water and then re-glove.

• If allergic reaction occurs, seek medical attention as needed.

3M ESPE MSDSs can be obtained from www.mmm.com or contact your local subsidiary.

Precautions During Processing

• Seat temporary restorations using a eugenol-free product (e.g., with RelyX™ Temp NE, manufactured by 3M ESPE). The use of temporary cements containing eugenol may interfere with the polymerization of RelyX Unicem 2 Automix due to the final curing.

• Do not use hydrogen peroxide (H_2O_2) as the residues can adversely affect the adhesive strength and setting reaction of RelyX Unicem 2 Automix.

Pulp Protection

To avoid pulpal irritation, cover areas in close proximity to the pulp by applying small amounts of hard-setting calcium hydroxide material. To avoid irritation during the cementing of the restorations, the pulp protection should be applied prior to taking an impression for the final restoration.

Tooth Preparation

• Prior to final cementation, clean the prepared stump or the cavity thoroughly with pumice slurry, rinse with a water spray, and lightly air dry in only 2-3 bursts of water-free and oil-free air, or use cotton pellets to dry it off.

• The cavity should be just dry enough that its surface has a slightly glossy appearance. As is the case with any permanent cement, overdryening can lead to post-operative sensitivity.

Do not use substances such as desensitizers, disinfectants, astringents, dentin sealants, sealing agents containing EDTA, etc., after the final cementation with the exception of the cementing of the relevant national professional associations. These substances must be observed for each application.

Preparation of Root Canals

• Treat the root canal endodontically as usual (root canal filling with gutta-percha and removal of the root canal filling, leaving 4 mm of gutta-percha apically).

• Clean the root canal with a 2.5-2.5% solution of sodium hypochlorite ($NaOCl$).

• Rinse immediately with water and dry with paper points.

• We recommend the use of a rubber dam during the cementation of posts.

Preparation for Maryland and Inlay/Onlay Bridges

Abutment teeth must have an adequate enamel surface for bonding. They should be healthy or only slightly restored and the periodontal conditions should be good. It is the sole responsibility of the dentist to ensure proper selection of indication and placement. The indications of the relevant national professional associations must be observed for each application.

• Prepare retentive elements, such as singular rings and/or approval grooves.

• Use a rubber dam and keep the cavity free of any contamination during cementation.

• Etch the enamel surface of the cavity with 37% phosphoric acid for 15-20 sec. Then rinse thoroughly with water and dry with water-free and oil-free air. In the case of exposed dentin make sure to selectively etch the enamel to avoid post-operative sensitivity.

Preparatory Measures

• Remove the temporary restoration and thoroughly clean the tooth surface of any residue from the temporary cement.

• Make a final insertion of the final restoration and check its fit and contact points.

• If a low viscosity silicone is used for the trial fit, the silicone residue must then be removed completely.

- For glass ceramic restorations, do not check the occlusion until the restoration has been cemented. Breakage could occur if done prior to being cemented in place.
- Avoid any contamination whatsoever of the surfaces to be treated during pre-treatment and until the final cementation.

Pre-treatment of Restorations

Please follow the instructions for use of the restoration material. In the absence of deviating instructions, we recommend the following procedure:

• Blast the restoration surface to be luted with aluminum oxide $\leq 40\text{ }\mu\text{m}$.

• Clean the blasted surface with alcohol and dry it with water-free and oil-free air.

For Maryland and inlay/onlay bridges, please refer to "Pre-treatment of Maryland and Inlay/Onlay bridges".

Pre-treatment of Etchable Glass Ceramic Restorations

Please follow the instructions for use of the restoration material. If the manufacturer has not provided deviating instructions, we recommend the following procedure:

• Use hydrofluoric acid to etch the glass ceramic restoration surface to be luted.

• Rinse thoroughly with water for 15 seconds and dry with water-free and oil-free air.

For RelyX and inlay/onlay bridges, please refer to "Pre-treatment of Maryland and Inlay/Onlay bridges".

Pre-treatment of Inlay/Onlay Brackets

Please follow the instructions for use of the restoration material. If the manufacturer has not provided deviating instructions, we recommend the following procedure:

• Use RelyX and inlay/onlay brackets for the bracketing of the inlay/onlay.

• Use hydrofluoric acid to etch the inlay/onlay brackets.

• Rinse thoroughly with water for 15 seconds and dry with water-free and oil-free air.

For RelyX and inlay/onlay bridges, please refer to "Pre-treatment of Maryland and Inlay/Onlay bridges".

Pre-treatment of Zirconia and Aluminum Oxide Ceramic Restorations

Please follow the instructions for use of the restoration material. If the manufacturer has not provided deviating instructions, we recommend the following procedure:

• Alternative 1:

• Blast the restoration surface to be luted with the micro-blasting device Cojet™ Prep and the blast-cleaning agent Cojet Sand™ from a distance of 2-10 mm and vertically to the surface for 15 sec, see instructions for use for Cojet Prep and Cojet Sand; both products are manufactured for or by 3M ESPE, respectively.

• Clean the blasted surface with alcohol and dry it with water-free and oil-free air.

For RelyX and inlay/onlay bridges, please refer to "Pre-treatment of Maryland and Inlay/Onlay bridges".

Pre-treatment of Composite Restorations

Please follow the instructions for use of the restoration material. If the manufacturer has not provided deviating instructions, we recommend the following procedure:

• Clean the post with alcohol and dry it with water-free and oil-free air.

When using RelyX Fiber Post, manufactured for 3M ESPE, application of a silane is not required. Other fiber-reinforced posts should be treated in accordance with the relevant instructions for use.

Pre-treatment of Implant Abutments

Please follow the recommendations for the relevant restorative material (e.g., metal, zirconia).

Pre-treatment of Maryland and Inlay/Onlay Bridges

Metal, Zirconia and Aluminum Oxide Surfaces:

• Coat (silicate) the restoration surface to be luted with the micro-blasting device Cojet™ Prep and the blast-cleaning agent Cojet Sand™ from a distance of 2-10 mm and vertically to the surface for 15 sec, see instructions for use for Cojet Prep and Cojet Sand.

• Blow away any residues of the blasting agent with water-free and oil-free air.

For etchable glass ceramics please refer to "Pre-treatment of Etchable Glass Ceramic Restorations".

Time

The processing and setting times depend on the ambient and oral temperature.

The times shown are based on conditions relevant for practice. As is the case with every composite cement, the setting of RelyX Unicem 2 Automix cement slows down significantly at room temperature.

RelyX Unicem 2 Automix is a dual-curing cement and therefore also sensitive to natural or artificial light. The working time is significantly reduced during application under operating lights!

Storage and Stability

After removal from the pouch, use the product within 6 months and before the lapse of the expiration date. Constantly high humidity accelerates the setting and must be avoided.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warnings

3M ESPE warrants this product will be free from defects in material and manufacture.

3M ESPE MAKES NO OTHER WARRANTIES, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.

User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE's sole obligation shall be repair or replacement of the 3M ESPE product.

Limitation of Liability

Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence, or strict liability.

Pre-treatment of Composite Restorations

Please follow the instructions for use of the restoration material. If the manufacturer has not provided deviating instructions, we recommend the following procedure:

• Clean the post with alcohol and dry it with water-free and oil-free air.

When using RelyX Fiber Post, manufactured for 3M ESPE, application of a silane is not required. Other fiber-reinforced posts should be treated in accordance with the relevant instructions for use.

Pre-treatment of Inlay/Onlay Brackets

Metal, Zirconia and Aluminum Oxide Surfaces:

• Coat (silicate) the restoration surface to be luted with the micro-blasting device Cojet™ Prep and the blast-cleaning agent Cojet Sand™ from a distance of 2-10 mm and vertically to the surface for 15 sec, see instructions for use for Cojet Prep and Cojet Sand.

• Blow away any residues of the blasting agent with water-free and oil-free air.

For etchable glass ceramics please refer to "Pre-treatment of Etchable Glass Ceramic Restorations".

Time

The processing and setting times depend on the ambient and oral temperature.

The times shown are based on conditions relevant for practice. As is the case with every composite cement, the setting of RelyX Unicem 2 Automix cement slows down significantly at room temperature.

RelyX Unicem 2 Automix is a dual-curing cement and therefore also sensitive to natural or artificial light. The working time is significantly reduced during application under operating lights!

Storage and Stability

After removal from the pouch, use the product within 6 months and before the lapse of the expiration date. Constantly high humidity accelerates the setting and must be avoided.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warnings

3M ESPE warrants this product will be free from defects in material and manufacture.

3M ESPE MAKES NO OTHER WARRANTIES, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.

User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE's sole obligation shall be repair or replacement of the 3M ESPE product.

Limitation of Liability

Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence, or strict liability.

Pre-treatment of Composite Restorations</

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3M ESPE, Cojet, Lava, RelyX and Sof-Lex are trademarks of 3M or 3M ESPE AG.